

InBody

Lookin'Body Web

Payment

Payment system

There are several compelling reasons for our decision to transition from PayPal to Stripe as our preferred payment processing module:

1.Enhanced User Experience: With Stripe's modern and user-friendly interface, we believe our customers will have a seamless and hassle-free payment experience, leading to increased customer satisfaction.

2.Robust Feature Set: Stripe offers a wide range of features that can greatly benefit our business, including automated payments, subscription management, and support for various payment methods. These features will allow us to streamline our payment processes, automate recurring payments, and provide more flexibility to our customers, ultimately improving our overall efficiency and effectiveness.

3.Flexibility and Customization: This includes options for customizable checkout experiences, flexible pricing models, and advanced reporting and analytics, which can provide valuable insights for our business decision-making.

In conclusion, We are confident that this change will contribute to our overall success and drive better results for our customers and our organization.

Subscription

Devices 10 Devices in Use

If you would like to add InBody equipment, please contact customer service.

	Serial No.	Model
1	570HOON01	InBody570
2	570HOON123	InBody570
3	570HOON577	InBody570
4	DEMOI3002	InBodyBWA20

Staff Accounts 1/2 Seats in Use

After purchasing staff accounts, you can enter detailed information in the ['Staff Account Management'](#) tab.

Standard
included
2

Additional
0

–

+

Total
2

	ID	Email	
1	nedstaff1	example@domain.com	
2	(Available)		

Detail

Payment Period

☒ 6 Month ☐ 1 Year

	Devices	Staffs	Price
Standard	1	2	EUR €150
Added Devices	9		EUR €1350
Added Staffs		0	EUR €0
Total	10	2	EUR €1500

Pay

Change in payment UI.

You customer can add staff account at this page also. But it requires proration payment after change.

Detail

Payment Period

☒ 6 Month ☐ 1 Year

	Devices	Staffs	Price
Standard	1	2	EUR €150
Added Devices	9		EUR €1350
Added Staffs		0	EUR €0
Total	10	2	EUR €1500

Pay

Customer would be redirected to the payment page of Stripe by clicking “Pay” button below.

It would be changed to “Update” button after the first payment.

 InBody Europe B.V.

Try Standard and 1 more

629 days free

€1,815.00 every 6 months thereafter

Standard	629 days free
Quantity 1	€150.00 thereafter /6 months

Device	629 days free
quantity 9	€1,350.00 thereafter /6 months

subtotal	€1,500.00
-----------------	------------------

[Add promo code](#)

VAT (21%) ⓘ	€315.00
-------------	---------

Total after trial	€1,815.00
--------------------------	------------------

Total payment on the day	€0.00
---------------------------------	--------------

Enter payment details

email

payment method

 card

 **SEPA**
SEPA direct debit

card information

1234 1234 1234 1234

MM/YY

Security code (CVC)

card holder

country or region

Netherlands

☐ Store my information securely for one-click checkout
Checkout faster with InBody Europe BV and wherever links are accepted.

☐ Purchased for business purposes

Start Trial

At the end of the trial, you will be charged €1,815.00 every 6 months from 31 December 2024 . You can cancel at any time before that.

 InBody Europe BV will donate **0.5% of your subscription fee** towards removing CO₂ from the air .

When click pay button it would make this popup.

After the days left for the expire(in the backoffice), it would automatically charge the certain amount of money through registered card or bank info.

 InBody Europe B.V.

Try Standard and 1 more

629 days free

€1,815.00 every 6 months thereafter

Standard

Quantity 1

629 days free

€150.00 thereafter /6
months

Device

quantity 9

629 days free

€1,350.00 thereafter /6
months

subtotal

€1,500.00

[Add promo code](#)

VAT (21%) ⓘ

€315.00

Total after trial

€1,815.00

Total payment on the day

€0.00

Enter payment details

email

payment method



SEPA

SEPA direct debit

card information

1234 1234 1234 1234	
MM/YY	Security code (CVC)

card holder

country or region

Netherlands ▼

☐ **Store my information securely for one-click checkout**

Checkout faster with InBody Europe BV and wherever links are accepted.

☐ **Purchased for business purposes**

Start Trial

At the end of the trial, you will be charged €1,815.00 every 6 months from 31 December 2024 . You can cancel at any time before that.



InBody Europe BV will donate **0.5% of your subscription fee** towards removing CO₂ from the air .

Customers can register their own vat nubmer

 InBody Europe B.V.

Try Standard and 1 more

629 days free

€1,815.00 every 6 months thereafter

Standard

Quantity 1

629 days free

€150.00 thereafter /6
months

Device

quantity 9

629 days free

€1,350.00 thereafter /6
months

subtotal

€1,500.00

[Add promo code](#)

VAT (21%) ⓘ

€315.00

Total after trial

€1,815.00

Total payment on the day

€0.00

Enter payment details

email

payment method



SEPA
SEPA direct debit

card information

1234 1234 1234 1234



MM/YY

Security code (CVC)

card holder

country or region

Netherlands



☐ **Store my information securely for one-click checkout**

Checkout faster with InBody Europe BV and wherever links are accepted.

☐ Purchased for business purposes

Start Trial

At the end of the trial, you will be charged €1,815.00 every 6 months from 31 December 2024 . You can cancel at any time before that.



InBody Europe BV will donate **0.5% of your subscription fee** towards removing CO₂ from the air .

For 20 months free usage of the beginning users, Promotion code from accounting team is required.

How can accounting team make promotion code?

The screenshot shows the nl.inbody dashboard. The top navigation bar includes links for Home, Payments, Balances, Customers, Products (highlighted), Billing, Reports, Connect, and More. A search bar contains the word 'coupon'. On the right, there are links for Create, Help, and a user profile icon. Below the navigation bar, the 'Products' section is active, with sub-tabs for All products, Coupons (selected), Shipping rates, Tax rates, and Pricing tables. Action buttons for Filter, New, and Export are visible on the right.

The 'Create a coupon' form is displayed. It includes a title 'Create a coupon' and a subtitle 'Coupons can be used to discount invoices, subscriptions, or entire customer accounts.' The form has several sections: 'Name' with a text input field containing 'First purchase discount'; 'ID' with a radio button for 'Optional' and an empty text input field; 'Type' with radio buttons for 'Percentage discount' (selected) and 'Fixed amount discount'; 'Percentage off' with a slider set to 100%; 'Apply to specific products' with an unchecked radio button; 'Duration' with a dropdown menu set to 'Multiple months'; 'Number of months' with a text input field containing '20'; and 'Redemption limits' with two unchecked checkboxes: 'Limit the date range when customers can redeem this coupon' and 'Limit the total number of times this coupon can be redeemed'. A link 'Learn more ->' is provided at the bottom.

As control on price is on accounting team, many other price options would be available with this coupon.

The screens are for 20 months free subscription. However, it would not change the backoffice expire date -> please contact person in charge.

UK : Adam

NL+other EU : Michiel

DE : not decided yet

Dashboard

Report

Member List

Health Report

Email/Chat

Print

Edit ▶

Recommendation

Device Management

Staff Management

Setup

Payment

 Tutorial: Getting Sta...

 FAQ

 Chat

 Contact Us

After completing your work in the new tab, please refresh the page to reflect the new status.

Refresh Page

Click “Refresh page” after payment.

InBody Age

InBody Age


Imagine knowing your body's true age - not just the number of years you've been alive, but the actual age of your body's health. Introducing InBody Age, a cutting-edge health assessment tool that provides a revolutionary approach to understanding your body's true age and overall wellness.

Unlike traditional age measurements based solely on chronological years, InBody Age goes beyond the surface to provide a comprehensive assessment of your body's internal health. Using advanced bioelectrical impedance analysis (BIA) technology, InBody Age measures multiple parameters to calculate your InBody Age, providing you with a holistic view of your body's health and helping you make informed decisions to optimize your well-being.

In the Health Report tab, you would be able to see the InBody Age.
Supporting devices are only 770 and 970.

Currently it's only available for the measurements after the update.
There would be update for old measurements later.

The language is based on setup of LB web not the result paper.

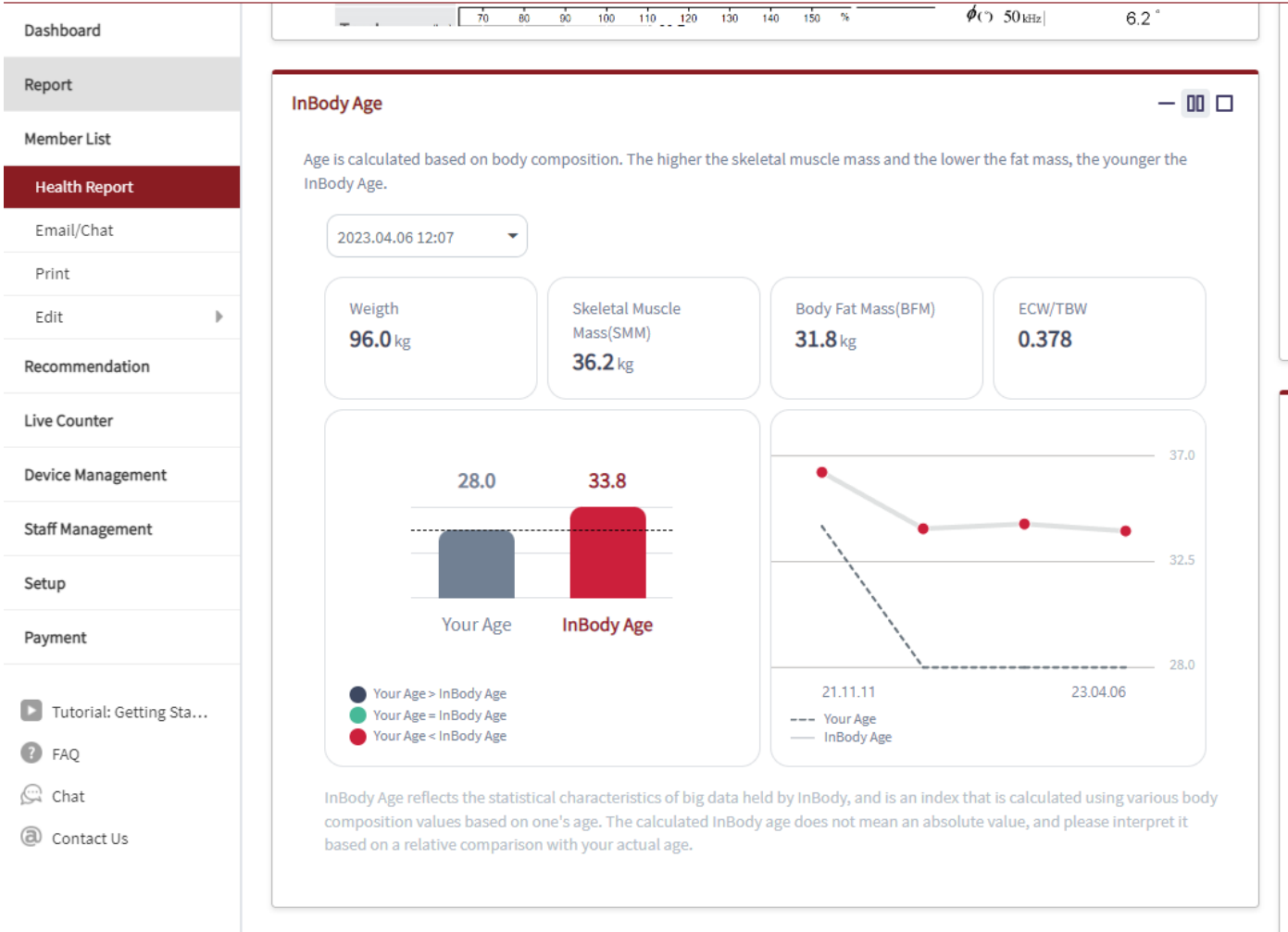

Recommendation

Recommendation

Our recommendation of corresponding supplements is designed to simplify the supplement selection process, save customers' time and effort in researching and comparing various products.

Also, it provides customers with targeted solutions to support members health goals. It would help them to add another business model on their business so that they can collaborate with supplements company.

Dashboard

Report

Member List

Health Report

Email/Chat

Print

Edit

Recommendation

Device Management

Staff Management

Setup

Payment

 Tutorial: Getting Sta...

 FAQ

 Chat

 Contact Us

TUTORIAL

Welcome!

Thank you for choosing LookinBody!

Create Staff/Employee Accounts

Connect InBody

Upload Member List

☐ Do Not show Again

TUTORIAL LIST

- Create Staff/Employee Accounts
- Connect InBody
- Upload Member List
- InBody Test and View InBody Results
- Organize Members into Groups
- Upload Previous Data

Two ways to access,

1. Navigation bar "Recommendation"
2. Access setup and click Nutrition

Dashboard

Report

Member List

Health Report

Email/Chat

Print

Edit

Recommendation

Device Management

Staff Management

Setup

Payment

 Tutorial: Getting Sta...

 FAQ

 Chat

 Contact Us

Setup

02. Results Sheet Types/ Paper Types

03. Outputs/ Interpretations for Result Sheets

04. Reference Range

05. N/A

06. Recommended Locations for InBody Units

07. Home Use InBody and InBodyBAND Data Display

LookinBody Data Management

01. Export Data as Excel

02. Import Group Registration Data as Excel

03. N/A

04. N/A

05. Data Importation

06. Temporary Data

07. Import Data from Previous LookinBody

Recommendation

Nutrition

Account Settings

01. Two-factor authentication

02. Check User Logs



Dashboard

Report

Member List

Health Report

Email/Chat

Print

Edit

Recommendation

Device Management

Staff Management

Setup

Payment

Tutorial: Getting Sta...

FAQ

Chat

Contact Us

Setup

02. Results Sheet Types/ Paper Types

03. Outputs/ Interpretations for Result Sheets

04. Reference Range

05. N/A

06. Recommended Locations for InBody Units

07. Home Use InBody and InBodyBAND Data Display

LookinBody Data Management

01. Export Data as Excel

02. Import Group Registration Data as Excel

03. N/A

04. N/A

05. Data Importation

06. Temporary Data

07. Import Data from Previous LookinBody

Recommendation

Nutrition

Account Settings

Recommendation

Nutrient recommendation is provided based on InBody results in Health Report.
You can recommend products according to the condition of each body composition.



Registered

4

Recommended

5

Go to main

Setting

Products

Tip

Good! Now, please recommend the registered products according to the body composition category.

Go

Product List

Product Name

Total 4 products

Delete

Add



Starred

Image

Name



test item



test set

With option 2.(through the setup), you would be able to access the page to setup the supplements.

In this page, click the Products and add buttons which are highlighted.

Account Settings

1

“Bulk registration is also an option”

Product Information

Name *

Image

Attach

Description *

Nutrients

Arginine

Bifidobacterium

BCAA

Calcium

Carnitine

Cancel

Save

The image above is when you choose to upload one product.

Product List

Product Name

Total 57 products

Delete

+

Add

☐	Starred	Image	Name
☐	★		Dymatize
☐	★		MCT-Oil
☐	★		Multi-Source Omega-3
☐	★		Forza Burn
☐	★		Merk
☐	★		Hypercor
☐	★		Laxodren
☐	★		Pro7ein Synthesis Gourmet
12345678			

Then it would be added like above.

- Dashboard
- Report
- Member List
- Health Report
- Email/Chat
- Print
- Edit
- Recommendation
- Live Counter
- Device Management
- Staff Management
- Setup**
- Payment
- Tutorial: Getting Sta...
- FAQ
- Chat
- Contact Us

Setup

- 02. Results Sheet Types/ Paper Types
- 03. Outputs/ Interpretations for Result Sheets
- 04. Reference Range
- 05. N/A
- 06. N/A
- 07. Home Use InBody and InBodyBAND Data Display

LookinBody Data Management

- 01. Export Data as Excel
- 02. Import Group Registration Data as Excel
- 03. N/A
- 04. N/A
- 05. Data Importation
- 06. Temporary Data
- 07. Import Data from Previous LookinBody

Recommendation

Nutrition

Account Settings

Recommendation

Nutrient recommendation is provided based on InBody results in Health Report.
You can recommend products according to the condition of each body composition.



Registered

57

Recommended

58

Go to main

Setting

Products

Setting

Recommendations are possible according to the body composition result outputs provided by the InBody device.

SMM Skeletal Muscle Mass	▲ Increase	7	Edit	☒
	— Maintain	0	Edit	☒
BFM Body Fat Mass	▲ Increase	9	Edit	☒
	— Maintain	0	Edit	☒
	▼ Decrease	5	Edit	☒
VFA	— Maintain	0	Edit	☒

Then, please access “Settings”

Dashboard

Report

Member List

Health Report

Email/Chat

Print

Edit

Recommendation

Device Management

Staff Management

Setup

Payment

Tutorial: Getting Sta...

FAQ

Chat

Contact Us

Setup

02. Results Sheet Types/ Paper Types

03. Outputs/ Interpretations for Result Sheets

04. Reference Range

05. N/A

06. Recommended Locations for InBody Units

07. Home Use InBody and InBodyBAND Data Display

LookinBody Data Management

01. Export Data as Excel

02. Import Group Registration Data as Excel

03. N/A

04. N/A

05. Data Importation

06. Temporary Data

07. Import Data from Previous LookinBody

Recommendation

Nutrition

Account Settings

Recommendation

Nutrient recommendation is provided based on InBody results in Health Report.
You can recommend products according to the condition of each body composition.



Registered

4



Recommended

4

Go to main

Setting

Products

Setting

Recommendations are possible according to the body composition result outputs provided by the InBody device.

SMM Skeletal Muscle Mass	▲ Increase	3	Edit	
	– Maintain	0	Edit	
BFM Body Fat Mass	▲ Increase	1	Edit	
	– Maintain	0	Edit	
	▼ Decrease	0	Edit	
VFA	– Maintain	0	Edit	

← Back

SMM ▲ Increase

Total 3 products

+ Add

Order	Starred	Image	Name
↑ ↓	☆		test set
↑ ↓	☆		test
↑ ↓	☆		teset

You would be able to add the products that you want to add through this page.

As example case clicked editing the products to “increase SMM”

You would be directed to page where you can add registered items helping “increase SMM”

In here you would be able to add

← Back

SMM ▲ Increase

Total 3 products

Order	Starred	Image	Name
↑ ↓	☆		test s
↑ ↓	☆		test
↑ ↓	☆		teset

+ Add

Dashboard

Report

Member List

Health Report

Email/Chat

Print

Edit

Recommendation

Device Management

Staff Management

Setup

Payment

Tutorial: Getting Sta...

FAQ

Chat

Contact Us

Setup

InBody Test Settings

01. N/A

02. Results Sheet Types/ Paper Types

03. Outputs/ Interpretations for Result Sheets

04. Reference Range

05. N/A

06. Recommended Locations for InBody Units

07. Home Use InBody and InBodyBAND Data Display

LookinBody Data Management

01. Export Data as Excel

02. Import Group Registration Data as Excel

03. N/A

04. N/A

05. Data Importation

06. Temporary Data

07. Import Data from Previous LookinBody

Recommendation

Nutrition

Renewal Date: 23.08.2023

InBody EU

SMM ▲ Increase

Total 4 products

☒

test item

☒

test set

☒

test

☒

teset

Cancel

Done

You can choose items to add from the items that you registered.

LookinBody

Dashboard

Report

Member List

Health Report

Email/Chat

Print

Edit

Recommendation

Device Management

Staff Management

Setup

Payment

Tutorial: Getting Sta...

FAQ

Chat

Contact Us

Setup

InBody Test Settings

01. N/A

02. Results Sheet Types/ Paper Types

03. Outputs/ Interpretations for Result Sheets

04. Reference Range

05. N/A

06. Recommended Locations for InBody Units

07. Home Use InBody and InBodyBAND Data Display

LookinBody Data Management

01. Export Data as Excel

02. Import Group Registration Data as Excel

03. N/A

04. N/A

05. Data Importation

06. Temporary Data

07. Import Data from Previous LookinBody

Recommendation

Nutrition

Renewal Date 23.08.2023

InBody EU

Recommendation

Nutrient recommendation is provided based on InBody results in Health Report.
You can recommend products according to the condition of each body composition.



Registered
4

Recommended
5

Go to main

Setting

Products

Setting

Recommendations are possible according to the body composition result outputs provided by the InBody device.

SMM Skeletal Muscle Mass	▲ Increase	4	Edit	☒
	– Maintain	0	Edit	☒
BFM Body Fat Mass	▲ Increase	1	Edit	☒
	– Maintain	0	Edit	☒
	▼ Decrease	0	Edit	☒
	– Maintain	0	Edit	☒

As you can see, it became 4

Dashboard

Report

Member List

Health Report

Email/Chat

Print

Edit

Recommendation

Live Counter

Device Management

Staff Management

Setup

Payment

Tutorial: Getting Sta...

FAQ

Chat

Contact Us

Setup

02. Results Sheet Types/ Paper Types

03. Outputs/ Interpretations for Result Sheets

04. Reference Range

05. N/A

06. N/A

07. Home Use InBody and InBodyBAND Data Display

LookinBody Data Management

01. Export Data as Excel

02. Import Group Registration Data as Excel

03. N/A

04. N/A

05. Data Importation

06. Temporary Data

07. Import Data from Previous LookinBody

Recommendation

Nutrition

Account Settings

Recommendation

Nutrient recommendation is provided based on InBody results in Health Report. You can recommend products according to the condition of each body composition.

Registered

57

Recommended

58

Go to main

Setting Products

Setting

Recommendations are possible according to the bod

SMM
Skeletal Muscle Mass

▲ Increase

→ Maintenance

BFM
Body Fat Mass

▲ Increase

→ Maintenance

Recommendation

Device Management

→ Maintenance

Setup

Payment

Tutorial: Getting Sta...

FAQ

Chat

Contact Us

Recommendation

You can recommend products according to the condition of each body composition.

Go to Setup

Search Name, ID, or Mobile No.

Latest

InBody Test date	ID	Name	Mobile No	
28.02.2023 16:12 (InBody 370S)	-pztldsoiztyav	moritz inbody test	01111331	View Result
28.02.2023 16:08 (InBody 370S)	1hz2o0wtzguwr	Moritz Schönhöfer	001111	View Result
27.02.2023 12:30 (InBody 370S)	-1qegi5gl0lzmng	Billy Nam		View Result
24.02.2023 10:32 (InBody 370S)	0614920306		0614920306	View Result
24.02.2023 10:28 (InBody 770)	0614920306		0614920306	View Result
24.02.2023 10:24 (InBody 970)	0614920306		0614920306	View Result
18.02.2023 10:59 (InBody 570)	-1qegi5gl0lzmng	Billy Nam		View Result

This added items would be suggested in the Recommendation button on navigation bar or Go to main button. Then click the “view result”

Recommendation Result

Print

🔍 Name, ID, or Mobile No.

0653630636 (35.0 / Female / 170.0cm) ▾

2023.04.07 09:33 (InBody 970) ▾

C Type

Overweight Morbid Obese Type

Weight 86.8 kg



SMM 28.4 kg



BFM 35.5 kg



Overweight Morbid Obese Type (C-type) appears when all the parameters are over the range but fat mass is extremely higher than the skeletal muscles, which is known as morbid obesity. It is expected to have fat-related issues (such as metabolic syn...

View More

SMM

(Skeletal Muscle Mass)

— Maintain

There are no products to recommend.
[Go to settings](#)

BFM

(Body Fat Mass)

▼ Decrease

InBody's Recommendation



Multi-Source Omega-3

The Department of Health and Human Services (HHS), the U.S. Department of Agriculture (USDA), the American Heart Association, and the American Dietetic Assoc...



Protein Gai7ner Synthesis


If you've been a regular at this gym thing for a while, you probably already know that to pack on serious size, you need to really up your carbohydrate and protein l...

View More

VFA

(Visceral Fat Area)

175.1 cm²

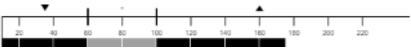

Please try these from your demo accounts.

Additional contents(upcoming) Live Count

Live Counter

This would not be updated soon, however, it is in the planner and checklist.

This would show two pages for counting the number of measurements which can be used in the exhibitions or congress as a dashboard.

- Dashboard
- Report
- Member List
- Health Report
- Email/Chat
- Print
- Edit ▶
- Recommendation
- Live Counter
- Device Management
- Staff Management
- Setup
- Payment
-  Tutorial: Getting Sta...
-  FAQ
-  Chat
-  Contact Us

Live Counter

Monitor InBody data statistics in real time.
Various screens are available. Run a Live Counter on the large display!

Live Counter List (2)



Seq	Create Date	Type	Event Name	Data Query Criteria	Usage	View Count	
2	2018.09.01	Normal Mode	1 Event 1	Inbody Inspection Date 2018.09.01 00:00 to present	Live Counter for the public	2	Live Counter run
1	2023.03.16	Normal Mode	InBody FIBO	Inbody Inspection Date 2023.03.16 11:46 to present	Live Counter for the public	6	Live Counter run

There are more

Who is the Female with the lowest Percentage Body Fat?





InBody Test Status

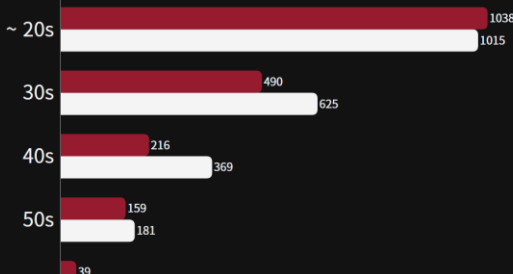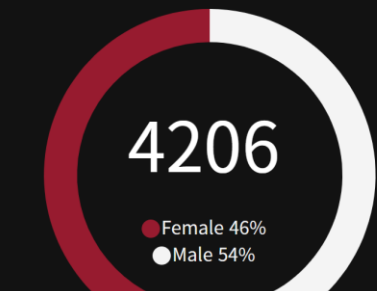

Who is the Female with the highest Skeletal Muscle Mass?

InBody Test Datetime	InBody Tester	SMM Rank
2023-04-07 09:33	Female 35 170.0cm	509 / 1
2023-04-06 13:20	Male 36 189.0cm	874 / 2
2023-04-06 13:15	Male 44 190.0cm	507 / 2
2023-04-06 12:07	Male 28 177cm	1756 / 2
2023-04-06 11:06	Male 30 171.0cm	1958 / 2
2023-04-06 10:01	Female 42 165cm	536 / 1
2023-04-05 13:54	Female 52 173cm	1333 / 1
2023-04-05 13:48	Female 57 168.0cm	1421 / 1941
2023-04-05 13:37	Male 58 177.5cm	1076 / 2265

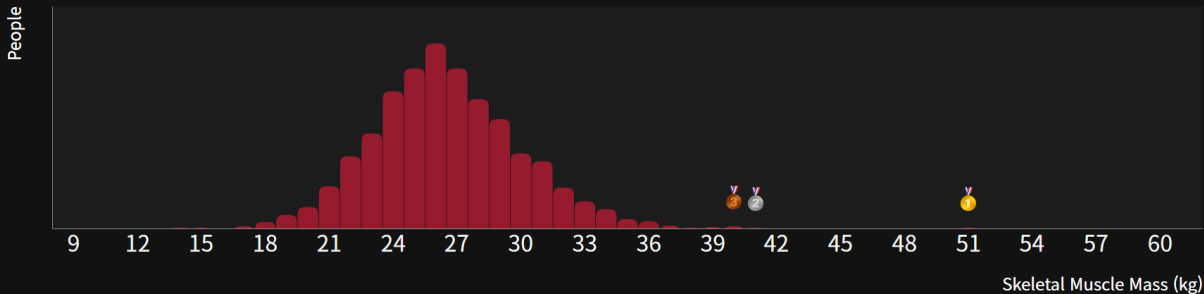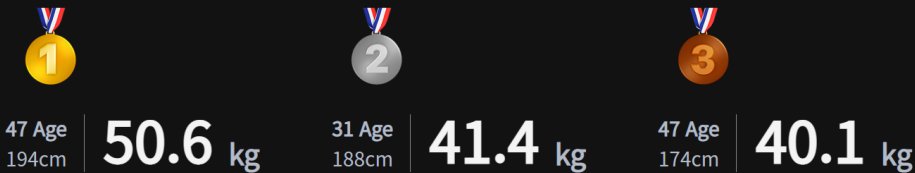

LIVE

89,704,923



InBody