




# Test Mode


## Loadcell test

Since firmware 0430 a Loadcell Test has been added, which needs to be PASS. It tests the reproducibility of the loadcell readings. You can use any consistent weight (including yourself if you can stay stable during the measurement). Otherwise a test weight or anything else available.

Click on option 5 'Loadcell test' :

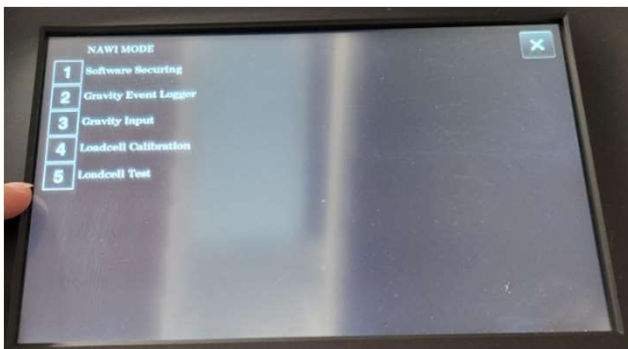

Place the weight once on the footplate, until it is read by the device :

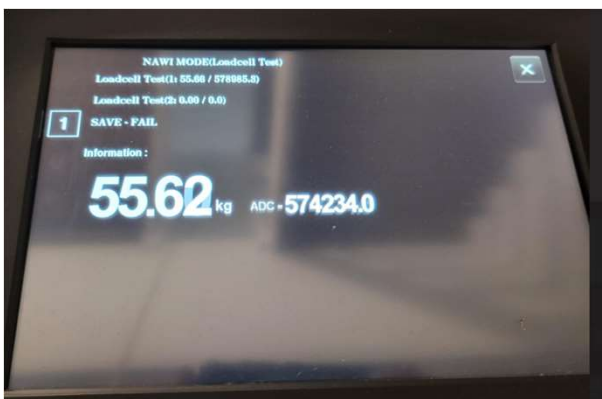



# Test Mode

## Loadcell test

Remove the weight and place it again :



Press 1. SAVE. If the weight reading are within margin, it will PASS. If they are not, it will FAIL.