

Clinical Change Letter

→ Digital Program Change in InBody770


1. Product Information

Model	Program Version	Date of Implementation
InBody770	770DM-0430	23.07.17


2. Details

1) LCD : Deletion of Mexican Growth Chart Standard in Spanish setting

- Decide to delete Mexican standard of Growth chart, which turns out to be the same as CDC2000
- CDC2000 will be automatically set as a default in Growth Chart

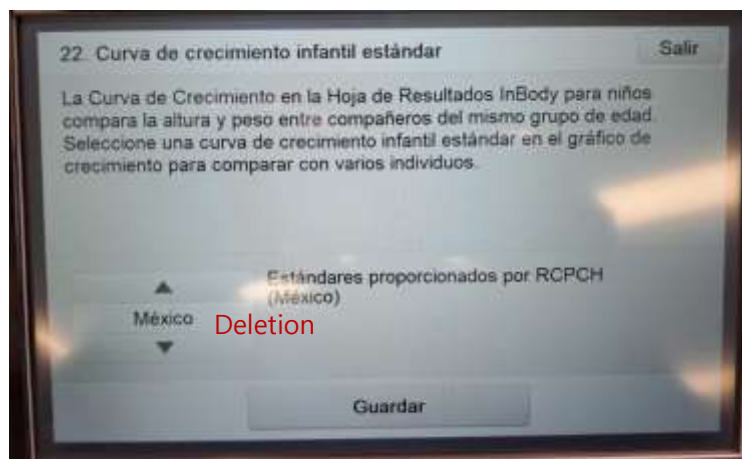

2) LCD : Update in Korean Growth Chart Standard: 2007 > 2017

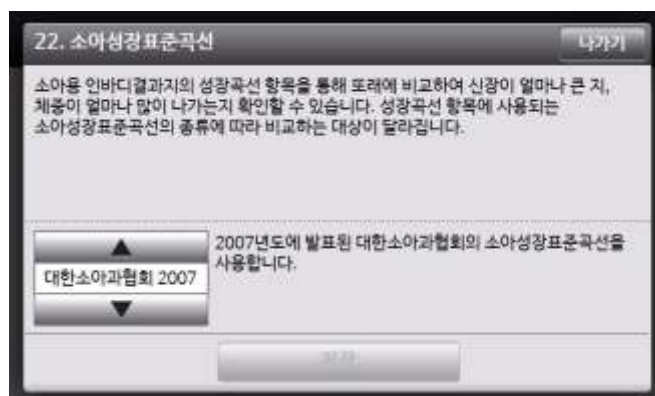

<Previous>



<0430 Version>

3) Change '0' to '-' in the value of Blood Pressure data in .csv in case of no BP data

- When Blood Pressure data is not existed, it was shown as '0' in .csv file → Changed to '-'

4) Addition of Evaluation Parameters in Open Protocol/API Results

- Nutritional Evaluation : Proteins, Minerals, Fat
- Obesity Evaluation : BMI, Percent Body Fat
- Body Balance Evaluation : Upper, Lower, Upper-Lower

5) Addition of normal range of Basal Metabolic Rate (BRM) in Open Protocol

- Normal Range of BMR is added in Open Protocol (except in Japanese version) due to demand from fields
(*Japanese version doesn't provide normal range for BMR from the device, so it is also not included in Results Sheets, .csv file and Open Protocol file.)

Changes	Entire languages			
	770		LB	
	Results	CSV	Results	CSV
1) Deletion of Mexican Growth Chart (The default chart will become CDC2002)	Amendment in both Device and LB setting (Spanish)			
2) Update in Korean Growth Chart Standard (2007 > 2017)	Amendment in both Device and LB setting (Koeran)			
3) Change '0' to '-' in the value of Blood Pressure data in .csv in case of no BP data	-	○	-	-
4) Addition of three evaluaion parameters in Open Protocol/API	Except USA Version			
5) Addition of normal range of Basal Metabolic Rate (BRM) in Open Protocol	Except Japanese Version			