

Document Number : IB-23-22

Released Date : 2023.07.24

Recipient : Domestic sales, Overseas sales

Reference : Quality Innovation Part, Production Part

Title : Digital Program (0430Ver) changed in InBody770

1. We are much appreciated for the hard work from branches and subsidiaries.

2. Change Classification (Reason)

- Clinical formula modification.
- Quality improvement.
- The requested changes from the subsidiary

3. Changed Details

[Clinical]

- **Modification of Clinical Formula (Clinical Letter will be sent separately for details)**

[Quality]

- Improvement in program for WiFi integration.
- Added a feature to delete automatic recording logs and removed the auto-save function.

[Requested by InBody Europe]

- German result sheet word correction.

[Requested by InBody Japan]

- Addition of evaluation criteria for Open Protocol/API.
- Modification of InBody to receive height/weight data from BSM370, along with 8 other changes.

[Requested by InBody USA]

- Addition of database recovery function, along with 1 other case.

[Requested by InBody China]

- Modification in notation of Chinese version.

[Requested by InBody Mexico]

- Changed the paper size to Letter to align with the product specifications for Mexico.

[Common]

- Added a member subscriptions and kiosk management function.
- Changed the Default setting for InBody(only for KOR) Pediatric Growth Curves to the '2017' version.
- Deleted an InBodyCare.com LCD image and comma in LookinBody word.
- Added a feature to delete automatic recording logs. and removed the auto-save function.
- Applied Security Code 071D

※ Refer to attachment 1 for detailed change history

4. Date of Implementation

Model	Program	S/N	Manufacture Date
InBody770 Gray	Digital Program(0430Ver)	I823000594	2023-07-17
InBody770 Brown	Digital Program(0430Ver)	C723000375	2023-07-17

5. Inventory Device Management

1) Device in Subsidiary and Distributer Inventory

- InBody770 inventory stock at each Subsidiary and Distributer location should be upgraded to Digital Program 0430Ver before release.

2) Research purpose device

- Upgrade the research purpose InBody770 to Digital Program 0430Ver.

Attachment 1

Digital Program (0430Ver) changed in InBody770

Departments	Changed Details
Clinical	[Modification of Clinical Formulas] - Please refer to the detailed information in the upcoming Clinical Letter.
Quality	[Improvement of program for WiFi integration] - Improved the issue where the device freezes on the ideal screen due to insufficient memory after connecting COMPAST 5G WiFi dongle. - Improved the issue where hidden networks were not being connected. - Resolved the problem where, even after selecting the SSID and entering the correct password, an error message stating "Incorrect password" would appear. - Reduced the time taken from WiFi search to connection from approximately 50 seconds to around 35 seconds, resulting in an improvement of approximately 15 seconds. - Improvement in the sensitivity of Wi-Fi signal displayed on the home screen, which was different from other products. - When multiple WiFi SSIDs with the same name (e.g., Mesh WiFi) are detected, only the one with the strongest signal strength will be displayed.
Quality	[Added a feature to delete automatic recording logs and removed the auto-save function] - Delete automatic recording logs when when initializing the event code in test mode. (Can be checked in the remaining memory on the first page of the Administrator Menu) - No longer automatically saves unnecessary logs.
EUROPE	[Correction of German Result Sheet Terminology] - Skeletspiermassa Index => Skelettuskel-Index
JAPAN	[Addition of Evaluation to Open Protocol/API] - Modified to transmit nutrition evaluation, obesity evaluation, and body balance evaluation values to the open protocol_Korean/Multilingual/Japanese
JAPAN	[Modification to receive height/weight data from BSM370 when connected to InBody products.] - When you touch the upper left corner of the stadiometer interlocking pop-up window 5 times as a hidden function, the phrase "Stadiometer is connected." is changed to "Stadiometer & Weight Scale is connected.", and the stadiometer image is changed to stadiometer&weight scale image. - Afterward, when measuring, the height/weight value is received from the stadiometer and output ont the screen, and the value is applied and measured during measurement.
JAPAN	[Addition of lock/unlock feature to the Open Protocol] - The existing lock command for the Open Protocol: '0' for a one-time unlock, '1' for lock setting. Addition of '2' command to unlock the device.
JAPAN	[Addition of impedance data to the Open Protocol] - Impedance data is transmitted through the Open Protocol after enabling the impedance hidden setting
JAPAN	[Administrator Menu 05. Even if age input is omitted in Age/Gender, if personal information is received from the cloud, it is modified to be applied as received information.]
JAPAN	[Corrected the WebStatus Error on the home screen to stop recording web logs.] - Improvements have been made to prevent unnecessary logging when the internet connection is lost.
JAPAN	[InBody Device UI Changes for WEB user] - Modification to InBody Device UI: Enable/Disable "Same as ID" Button through Hidden Function - Entering Test Mode > Setup > 04. Cloud Service,
JAPAN	[Aligned LB120 CSV with Device CSV] - Modified the display to "-" when blood pressure measurement is not taken.
JAPAN	[Modified the WiFi SSID input keyboard to allow input of special characters, same with password input keyboard]
JAPAN	[Modified the login screen after reboot to allow selection of "Yes" or "No" using the keypad] - Keypad 'Enter' is "YES", Keypad '<- ' is "NO"
USA	[Addition of DB Recovery Function] In the Administrator Menu, select "08. Data Backup/Restore/Combine." After entering the menu, touch the LCD screen in the bottom right corner five times. This will initiate the process to recover a corrupted database. When the process is complete, a ringing sound will be emitted.
USA	[Delete of the phrase "InBodyService.com"]
CHINA	[Modification of Version Labeling in Administrator Menu in Chinese] - When the language setting is set to Chinese, the version "IBX70-1.0" will be displayed as "IBX70C-1.0". - "IB770-2.0" and "IB570-2.0" will be changed to "IB770C-1.0" and "IB570C-1.0" respectively.
MEXICO	[Changed the paper size to Letter for Mexico] - Changed the paper size to Letter to align with the product specifications for Mexico.

Business parts (MEXICO)	[Removal of Growth Curves in "Mexico" Setting] - Removal of Growth Curves "Mexico standard" from the list as it is the same as CDC2000 standard.
Business parts	[Changed the Default setting for InBody(only for KOR) Pediatric Growth Curves to the '2017' version.]
Business parts (TAIWAN)	[Re-registration of Basal Metabolic Rate (BMR) Standard Range in Open Protocol] - Modified so that the basal metabolic rate standard is transmitted to an open protocol in all languages except US/Japanese.
SW parts	[Addition of Member Subscription and Kiosk-related Features] - In the Administrator Menu, if the internet connection is established, cloud usage is enabled, and the user is logged into the cloud, the corresponding features can be accessed. - In the 04. Cloud Service screen after logging in, pressing '0' on the keypad five times sets the device to European subscription mode (Count1 or CNT1). Pressing '0' five more times on the keypad switches it to American subscription mode (Count2 or CNT2). - After measuring body weight, the ID input screen is changed to a Pin Code input screen. After entering the Pin Code, the remaining measurement count is received from the server. - If the member has remaining measurement credits, the measurement will proceed. If there are no remaining credits, an error screen will be displayed. - The self mode has been disabled, and only the Professional mode is available for use.
-	[Applied Security Code 071D]
-	[Deleted an InBodyCare.com in LCD image and comma in LookinBody word]