



970 Rubber Installation






Rubber installation

For the installation of the rubbers underneath the 970 the following pieces are needed.

Method for Installing the InBody970 Foot Guard set

1) Attach the three Foot Guards to the lower cover

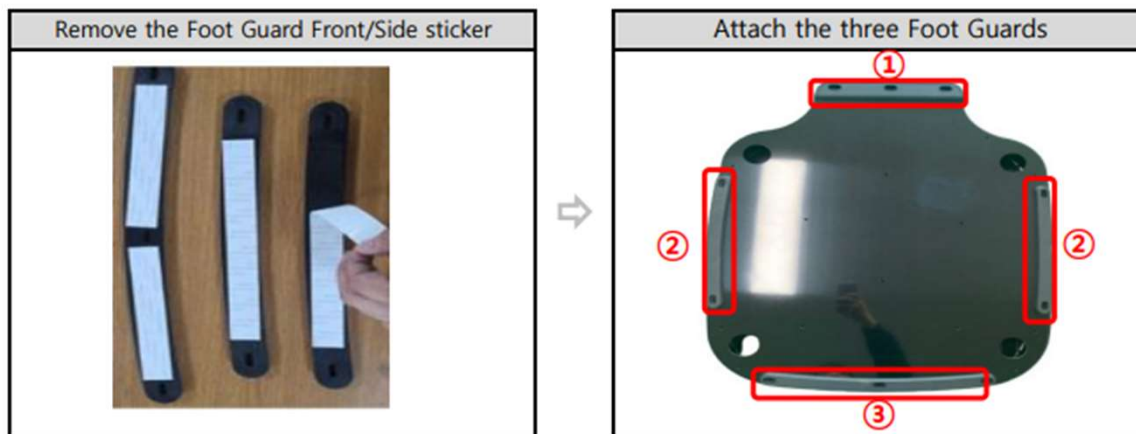

New material			
No	Code	Name	Quantity
①	I10404006	R_Foot Guard Lower InBody970	1EA
②	I10404007	R_Foot Guard Side InBody970	2EA
③	I10404005	R_Foot Guard Front InBody970	1EA

To attach the rubber, use the tape and original screw from part 2, 3 and 4, as well as a provided washers.

For part 1, use the tape, and longer screw and the provided washers