

Document Number : IB-24-13

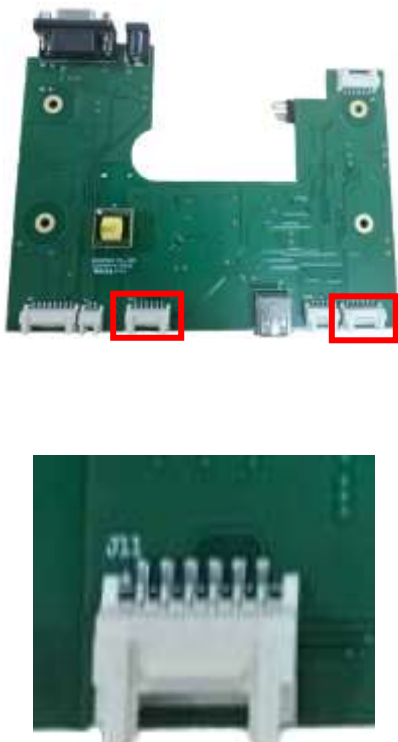
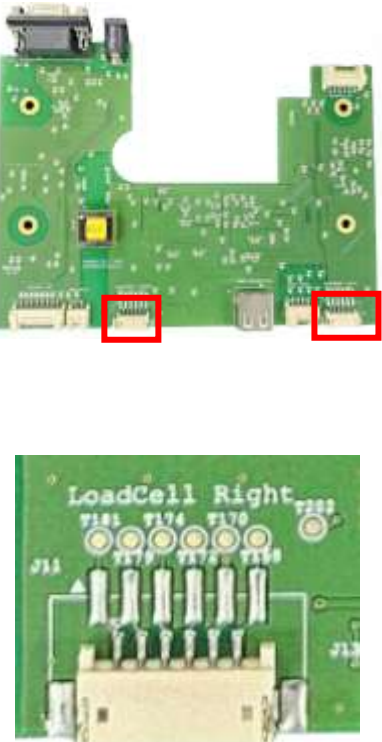
Released Date : 2024.05.27

Recipient : Domestic sales, Overseas sales

Reference : Quality Innovation Part, Production Part

Title : Change in Connector and Loadcell Housing of InBody120 IF Board

1. We are much appreciated for the hard work from branches and subsidiaries.
2. Change Classification (Reason)
 - To ensure compatibility of spare parts, Loadcell Housing and Board Connector have been modified.
3. Changed Details
 - 3.1 Modification in IF Board Connector

Item	Before	After
Photo		
Connector	20037WR-06	15001WR-06
Board Rev	D	E
Item Code	I8F310001	I8F310002
Item Name	Board IF 120	Board IF 120 (CE 4th)

Headquarter : InBody Bldg., 625, Eonju-ro, Gangnam-gu, Seoul 06106 Korea

Manufacturer : 15, Heugam-gil, Ipjang-myeon Seobuk-gu, Cheonan-si Chungcheongnam-do 31025 KOREA

Tel : 010-9909-2372

Fax : (041)581-3103

Department in charge : Quality Management Team of Quality Innovation Part

Person in charge : TaeGu Kang E-mail : kp016250@inbody.com

IB-105-035

InBody Co., Ltd

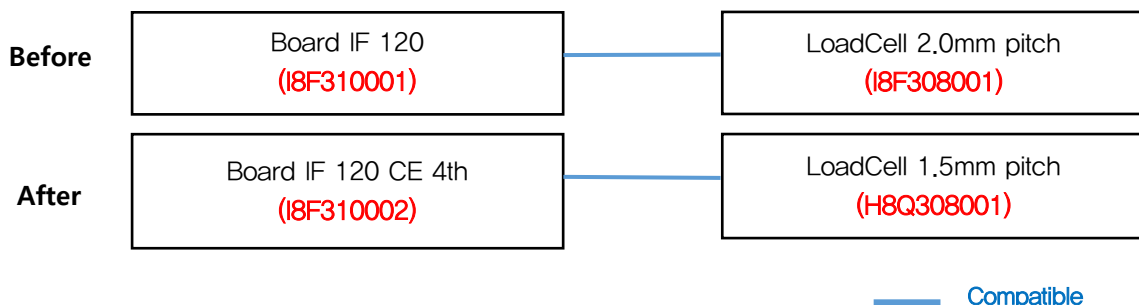
Rev.0

3.2 Modification in Loadcell Housing

Item	Before	After
Photo		
Housing	SMH200-06 (2mm pitch)	15001HS-06 (1.5mm pitch)
Item Name	[CS][Assy]LoadCell 2.0mm 120/H20B	[Assy]Loadcell 1.5mm 120/H20B/H30/H40/Fit
Item Code	I8F308001	H8Q308001

4. Compatibility

- Since the modified parts are not compatible with existing components, IF Board and Loadcell must be replaced as a set.
- The older model Board IF 120 and LoadCell 2.0mm can still be ordered.



5. Date of Implementation

Model	S/N	Manufacture Date
InBody120	F324001705	24/5/13

※ Same modification for the InBodyH20B will be implemented in the future. A separate notice will be provided when the changes are applied.