

Document Number : IB-24-22

Released Date : 2024.07.22

Recipient : Domestic sales, Overseas sales

Reference : Quality Part, Production Part

Title : Change in InBody380/580 Digital(0421ver) & LoadCell(C014ver) Program

1. We are much appreciated for the hard work from branches and subsidiaries.

2. Change Classification (Reason)

- Quality Improvement
- Modifying UI and Function

3. Changed Details

3.1 Digital Program (0421ver) Modification Details

- 1) Addition of NAWI mode Calibration Function for 4 LoadCells
- 2) Improvement of UI to prevent continuous error screens when the user steps off the foot case during the measurement.
- 3) Removal of Event Logger Initialize Function
- 4) Application of Feedback about NAWI Result Sheet and Enhancement of Firmware Upgrade Security to comply with NAWI certification
- 5) Modification of Multilingual Resources (English/France/Italia/Spain/Mexico/Thailand/Taiwan)

3.2 LoadCell Program (C014ver) Modification Details

- 1) Enhancement of Firmware Upgrade Security to comply with NAWI certification

※ Digital Program(0421ver) and LoadCell Program(C014ver) must be upgraded together for compatibility.

4. Date of Implementation

Model	Program	LoadCell Program	S/N	Manufacture Date
InBody380	0421ver	C014ver	P924001824	2024-07-16
InBody580	0421ver	C014ver	R624001573	2024-07-16
InBody380 NAWI	0421ver	C014ver	R324000001	2024-06-26
InBody580 NAWI	0421ver	C014ver	V224000001	2024-06-25